Hue Light

IMMUNITY CARE SYSTEMS

H-2000

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National Accredited Agency Test Report

Gas flow rate & generated components and amount analysis test

Verification No. (원본확인번호) 2905-8204-0979-6639

KRISS Korea Research Institute of Standards and Technology
287 Gajeong-ro, Yuseong-gu, Daejeon 305 380, Republic of Korea
Phone: +82-42-879-5222 Fax: +82-42-879-5205
Report No.: 2021-01361-001 Page 1 / 3 Pages

TEST REPORT

Applicant: Hae Light Co., Ltd.
60, Haean-ro, Oeanggyeong-gu, Oeanggyeong-do
Test Item: Molecular Hydrogen Inhalation Device
Manufacturer & Model: Hae Light Co., Ltd. / H-2000
Serial No.: HJLJ2-22-003
Date of Receipt: 21 December 2022 **Date of Test:** 26 January 2023

Description of Test
• **Test Name:** Gas analysis
• **Test Site:** KRISST Lab Mobile Lab On-site

Environmental Conditions
- **Temperature:** 20 ~ 22 °C - **Relative Humidity:** 40 ~ 60 %

Test Method:
- Test Procedure of Mixture and Pure Gas Measurement with Gas MS/CT-02-030-2000
- Test procedure of purity analysis of raw material(T-02-055-2017)

Test Results:
- Refer to the "TEST RESULTS" on the next page(s).
- **Measurement Uncertainty:**
- Refer to the "TEST RESULTS" on the next page(s).

General Information
- This certificate shall not be modified, partially reproduced or used in an advertisement or product promotion without the written approval of KRISST. Otherwise the legal action may be taken against violation.
- The above results are valid only for the sample provided by the applicant, and the name of the sample is provided by the applicant.

Tested by: Lee, Jin Guk
Date of Issue: 28 January 2023
Approved by: Jung, Jin Sang
President of KRISST

Verification No. (원본확인번호) 2905-8204-0979-6639

KRISS Korea Research Institute of Standards and Technology
287 Gajeong-ro, Yuseong-gu, Daejeon 305 380, Republic of Korea
Phone: +82-42-879-5222 Fax: +82-42-879-5205

H₂
O₂

67.38 cmol/mol⁽¹⁾, Relative expanded uncertainty (k=2): 0.5 %
32.44 cmol/mol, Relative expanded uncertainty (k=2): 1 %

1. Description of the test sample device
The test sample is a device that electrolyzes water to generate hydrogen and oxygen gas. It is discharged through one water trap without separating the generated hydrogen and oxygen gas (also called brown gas).

2. Principles of the test sample device
(1) Device name: Molecular Hydrogen Inhalation Device
(2) Model No.: H-2000
(3) S/N: HJLJ2-22-003

3. Product specifications
(1) Device name: Molecular Hydrogen Inhalation Device
(2) Model No.: H-2000
(3) S/N: HJLJ2-22-003

4. Test procedure
(1) Test sample were installed at KRISST laboratory (Fig. 1).
(2) A sampling kit and a 10 L sampling bag were installed at the outlet to collect the gas generated from the "Test sample".
(3) After operating the Test sample for about 5 hours or more, the generated gas was collected in

Verification No. (원본확인번호) 2905-8204-0979-6639

KRISS Korea Research Institute of Standards and Technology
287 Gajeong-ro, Yuseong-gu, Daejeon 305 380, Republic of Korea
Phone: +82-42-879-5222 Fax: +82-42-879-5205

H ₂	67.38 cmol/mol ⁽¹⁾ , Relative expanded uncertainty (k=2): 0.5 %
O ₂	32.44 cmol/mol, Relative expanded uncertainty (k=2): 1 %
N ₂	0.18 cmol/mol, Relative expanded uncertainty (k=2): 5 %
Ar	79.3 μmol/mol ⁽²⁾ , Relative expanded uncertainty (k=2): 10 %
CH ₄	N.D. ⁽²⁾ (Detection limit : 0.2 μmol/mol)
CO	0.02 μmol/mol, Relative expanded uncertainty (k=2): 50 %
CO ₂	13.3 μmol/mol, Relative expanded uncertainty (k=2): 3 %
THC(C ₂ -C ₆)	N.D.(Detection limit : 0.3 ppm/mol)
Flow rate	2.17 L/min, Relative expanded uncertainty (k=2): 3 %

6. Analytical instruments: Gas-MS(MAT 271), GC-FID/PID-methanator, Sampling kit, sampling bag, Integrated flowmeter(Shingawa W-NK-0.5A)

7. The measured value of the gas component is the mole fraction excluding water.

8. cmol/mol⁽¹⁾ and μmol/mol⁽²⁾ can be expressed as % and ppm, respectively.

Flow rate

2.17 L/min, Relative expanded uncertainty (k=2): 3 %

Fig. 1 Equipment configuration installed in KRISST at Hae Light Co., Ltd. (Continues to page 3)

12. CAUTION! Avoid ignition sources (fire, static electricity, etc.) at the inlet of the gas outlet of this test product (hydrogen generator). If an ignition source occurs, it may cause sparks or ignition. Therefore, when using the equipment, avoid a completely enclosed space and attention must be paid to the safety of the ignition sources.

13. This test result is issued at the given date. KRISST is not responsible for any change of the sample since the issued date. This test result is valid just for the given item.

14. This test report is no valid if it is copied.
(End of the Results).

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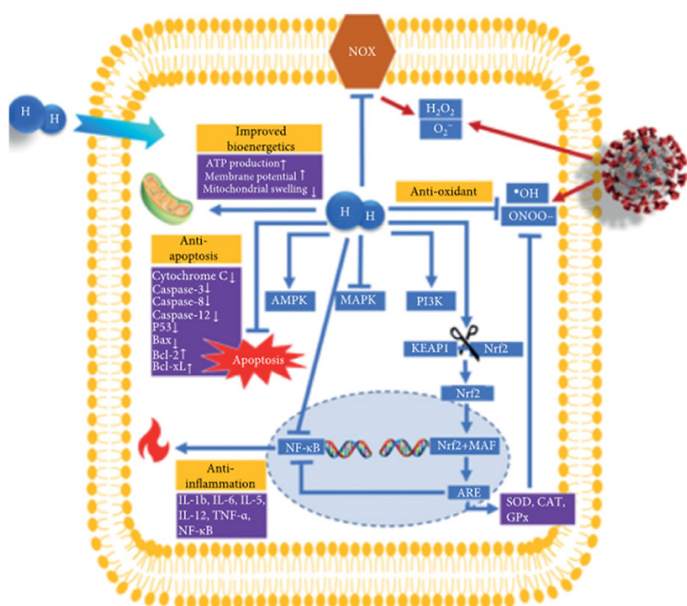
Mechanism

Possible mechanisms of alleviation properties of molecular hydrogen on COVID-19 patients.

Reactive Oxygen Species (oxidative stress & inflammation) due to COVID-19 virus -

OH: hydroxyl radical; O₂⁻: superoxide anion; ONOO⁻: peroxynitrite anion; H₂O₂: hydrogen peroxide; H₂: molecular hydrogen; NOX: nicotinamide adenine dinucleotide phosphate (NADPH) oxidase; AMPK: 5 adenosine monophosphate- (AMP-) activated protein kinase; MAPK: mitogen-activated protein kinase; PI3K: phosphatidylinositol 3-kinase; Keap1: Kelch-like ECH-associated protein 1; Nrf2: nuclear factor erythroid 2-related factor 2; MAF: small MAF protein; ARE: Nrf2-antioxidant response element; SOD: superoxide dismutase; CAT: catalase; GPx: glutathione peroxidase; P53: tumor protein; Bax: BCL2-associated X protein; Bcl-2: B-cell lymphoma 2 protein; Bcl-XL: B-cell lymphoma-extra-large protein; IL-12: interleukin 12; IL-1β: interleukin 1-beta; IL-6: interleukin 6; IL-8: interleukin 8; TNF-α: tumor necrosis factor α; NF-κB: nuclear factor-kappa-light-chain-enhancer of activated B cells.

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Wei-Jie Guan, Chun-Hua Wei, Ai-Lan Chen, Xiao-Cong Sun, Guang-Yun Guo, Xu Zou, Jin-Dong Shi, Pei-Zhen Lai, Ze-Guang Zheng, and Nan-Shan Zhong (2020)

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Duried Alwazeer, Franky Fuh-Ching Liu, Xiao Yu Wu, and Tyler W. LeBaron (2021)

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Hue Light Co., Ltd.